



wipro 3D



PRODUCT GUIDE

Wipro 3D SafeShield (Harness + Visor)

Product ID / Model No: W3D-FS-M1-V1


This document provides details of the product and instructions to assemble and use the product

Contents

1. Product Details
2. Items in Package
3. Assembly Instructions
4. Instructions for Cleaning
5. What to Expect & Not to Expect from this Product
6. Disclaimers



Wipro 3D SafeShield

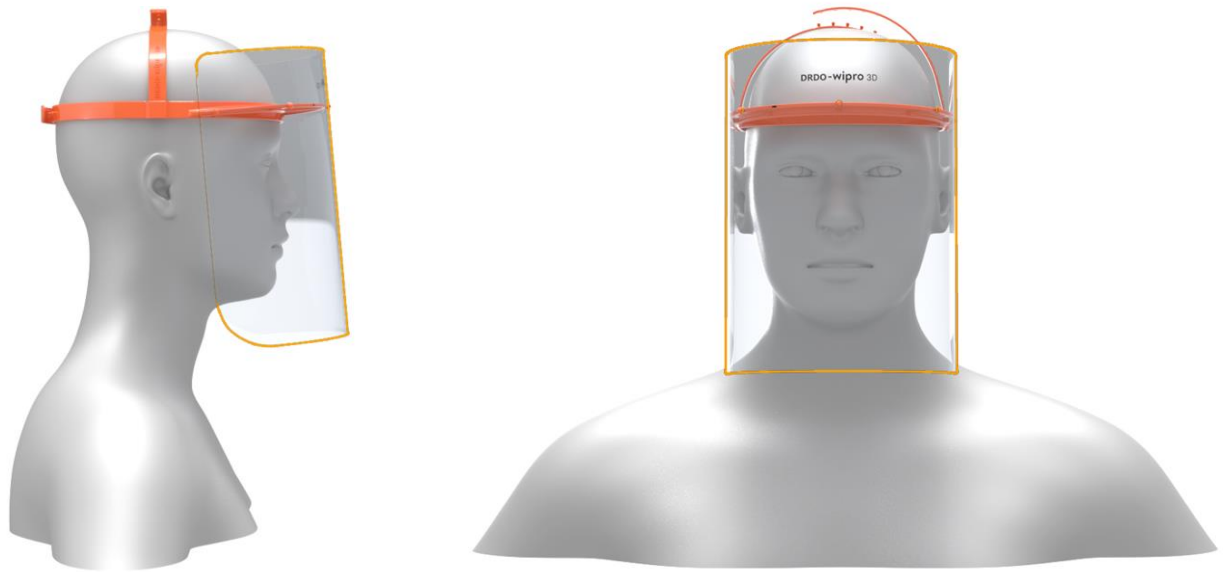

Typical Fitment of SafeShield

Product Details

- **Product Name:** Wipro 3D SafeShield
- **Application:** Personal Protective Equipment (PPE)
- **Protection from:** Respiratory emissions of COVID-19 infected person
- **Coverage:** Face including forehead, eyes, nose, mouth and partially neck
- **Size:** Harness - 255 mm x 190 mm x 100 mm (Approx.),
Visor - 250 mm (Height) x 330 mm (Circumference)
- **Sterility:** Delivered in non-sterile condition

Users

- Healthcare workers, doctors, nurses and paramedics
- Working and travelling individuals and delivery personnel
- Police and personnel involved in administrative measures
- Citizens who need to move out in public for essentials
- Shop floor and warehouse personnel
- In general, any individual who needs a broad non-specific protection against emissions.


HARNESS

Headrest with adjustable straps

Material: Polypropylene (PP)

Color: Orange (as per availability)

Sterilization: Ethylene oxide, Radiation, Steam

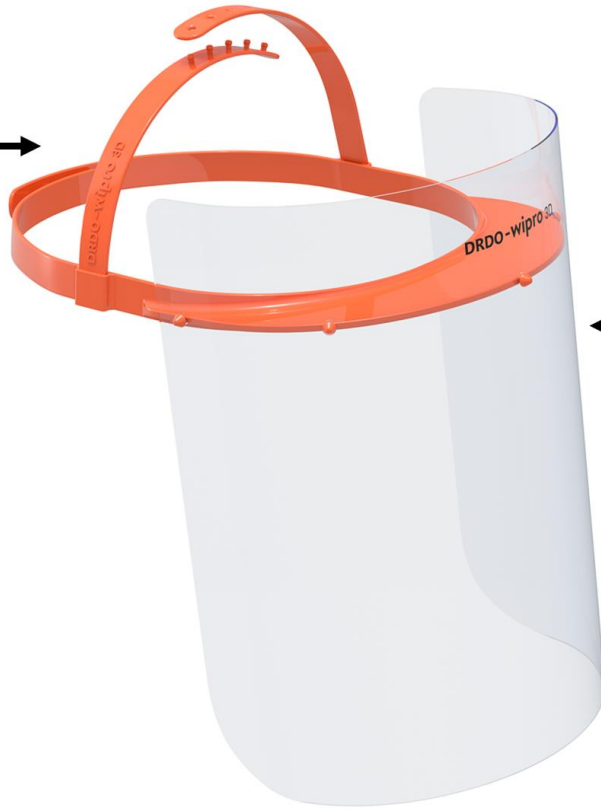


VISOR

Clear Protective Sheet


Material: Polyvinyl chloride (PVC)

Color: Transparent

Part Descriptions

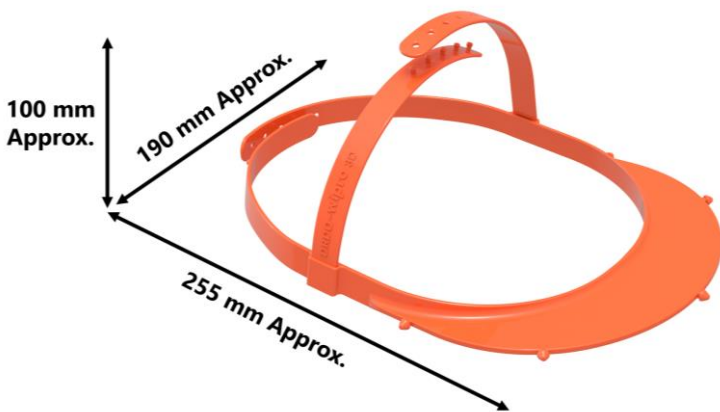


Assembled Harness

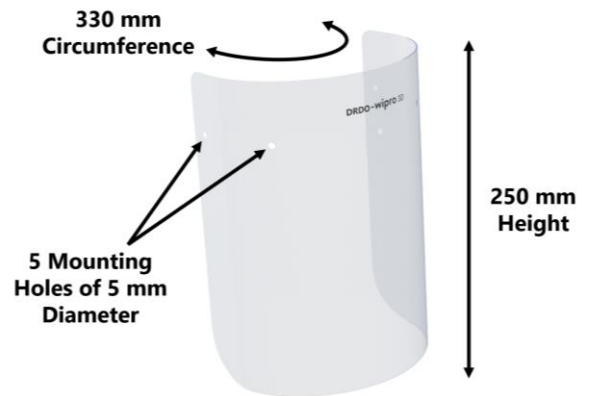

Visor

Part Dimensions


Items in Package

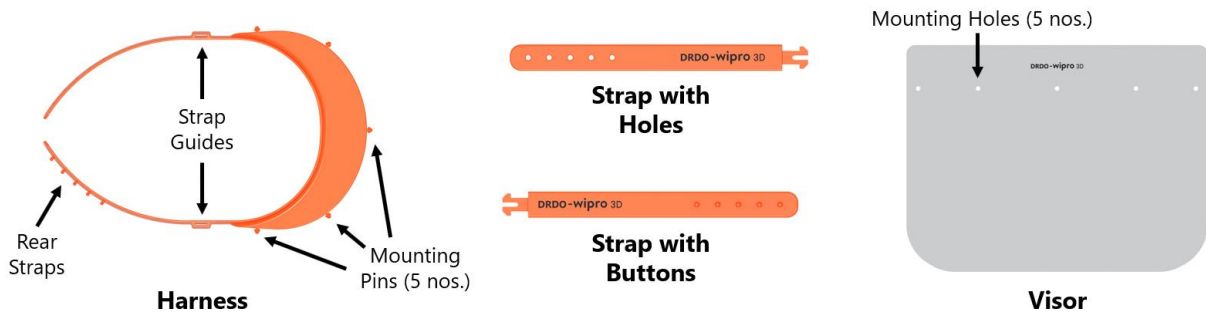

Assembly Instructions


Step 1

Place the **Strap with Buttons** on the **right** side of the Harness & place **Strap with Holes** on the **left** side of the Harness

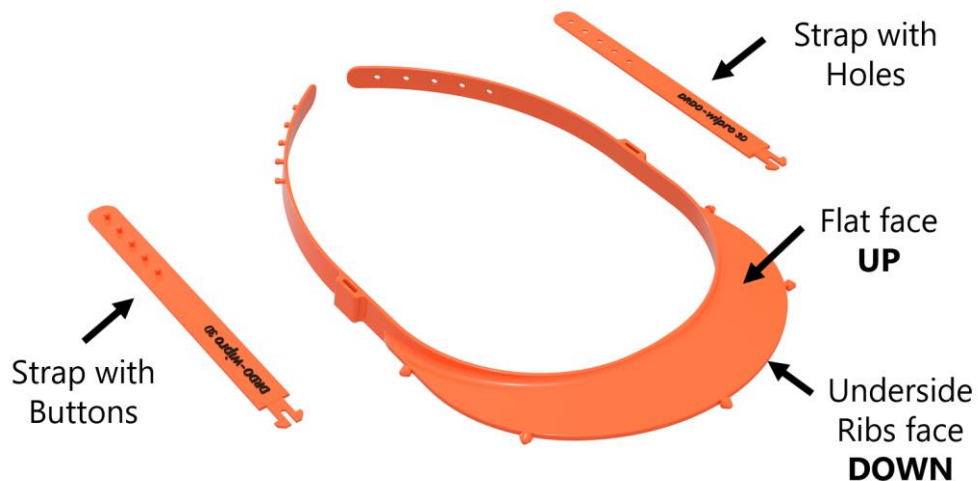

Step 2

Insert the **Straps into the Guides** provided on the sides of the Harness, until they **snap into place**

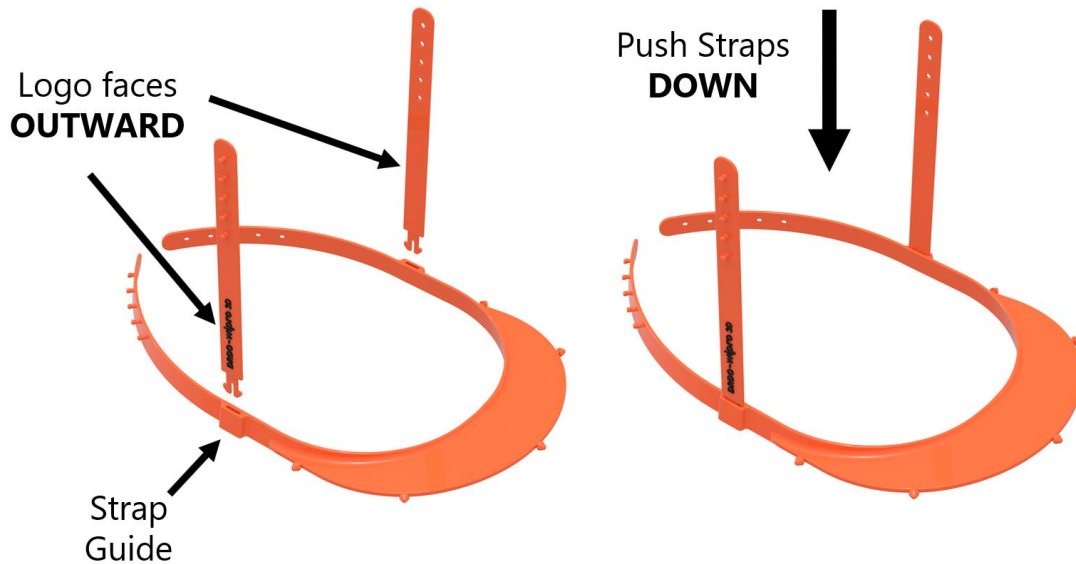


Step 3

Mount the flat **Visor** on one end of the Harness using the **Mounting Pin** then curve the Visor along the Harness while pushing the **Visor Mounting Holes over the Mounting Pins**



Step 4

Peel off the **protective cover on both sides of the Visor** sheet then adjust the Straps on the top and rear using the buttons for a comfortable fit

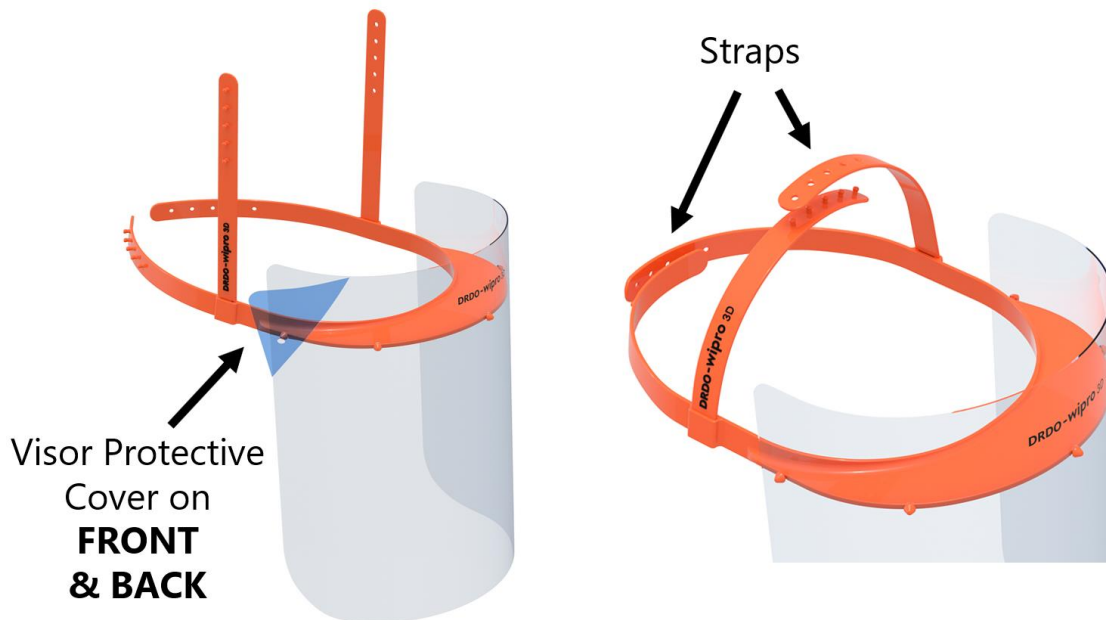

Instructions for cleaning the Harness

Clean Harness with Ethanol or Isopropanol with a soft cloth

Instructions for cleaning the Visor

In high virus-exposure environments, we recommend only one-time use of the Visor. This recommendation is for frontline healthcare workers who are handling COVID-19 cases.

However, for general purpose use the Visor can be cleaned with Ethanol or Isopropanol with a soft cloth and can be used till there is an issue with visibility due to scratches, dents etc.

What to expect from this product?

The SafeShield can be worn over the primary face mask to supplement shielding from the virus in “contact” environments and can be used standalone in low risk non-contact environments

What not to expect from this product?

This product is not a substitute for an N95 face mask.

Disclaimers

There may be slight variations in design, color, finish, dimensions and material based on constraints of manufacturing technology employed.

